

Screen shots are samples and may not be an exact match to your facility's device

**Welch Allyn Connex Spot Monitor (CSM)
User Quick Guide
ED Patient**

1. Power on CSM



2. Touch Caduceus ► Scan 3-4 Badge or Enter 3-4 ID and Network password (will be started out on screen)




3. Scan armband - Patient's partial name and account number display

4. Place BP cuff and press Start button ► Insert temp probe ► Place SpO2 sensor (captures pulse) ► Select box to right




The Modifiers screen displays ► Select desired modifiers ► Press OK

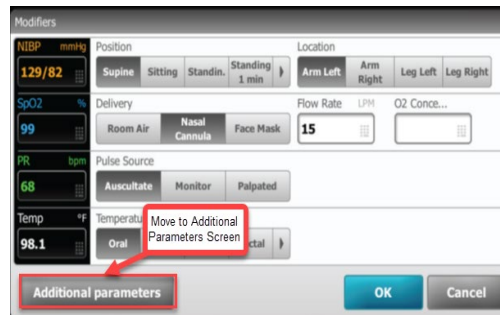

► Enter values ► Skip LOC field to prevent MEWS calculation for PRE ER and REG ER ► Press Next button

Note: There are no required fields

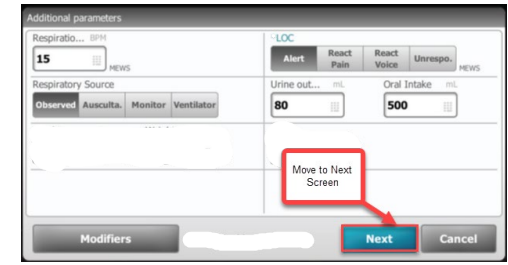

5. Skip any pop up messages for PRE ER and REG ER patient ► Press OK – Home screen displays ► Press Save button. MEWS must be calculated for OBS/In Patient Holds



**Welch Allyn Connex Spot Monitor User
Quick Guide**

ED HOLD/ADM IN or ADM INo Patient

- Power on CSM and follow steps for device use as for the ED patient.
Do not place patient in an Off Tracker status in Meditech.



- MEWS score is required for ED HOLD and OBS patients. ▶ Enter the Level of Consciousness (LOC) to calculate the MEWS Score.



- The MEWS score with a corresponding color and Instructions Message displays on the device. The score will be sent to Meditech. The score is calculated based on the following grid.

MEWS GRID							
Components	3	2	1	0	1	2	3
Systolic BP	<70	71-80	81-100	101-199		≥200	
Pulse Rate		<40	41-50	51-100	101-110	111-129	≥130
Temperature ©		<35		35-38.4		≥38.5	
Resp rate		<9		9-14	15-20	21-29	≥30
AVPU score				Alert	Reacting to Voice	Reacting to Pain	Unresponsive

MEWS Score	Rank and Color	MEWS WITH INSTRUCTIONS
0 – 1	NORMAL	None
2	LOW	If MEWS Score was a 0 from the previous recorded measure A. Licensed or non-licensed caregiver: -Notify RN of MEWS score B. Licensed caregiver: -Reassesses patients -Activate Rapid Response Team if clinical judgment deems necessary -Notify Physician per policy C. Charge Nurse: -Round on identified patients with primary nurse to determine if additional monitoring or intervention is needed.
3	MEDIUM	If the MEWS Score was 1 or less from the previous recorded measure A. Licensed or non-licensed caregiver: -Notify RN of MEWS score B. Licensed caregiver: -Reassesses patients -Activate Rapid Response Team if clinical judgment deems necessary – Notify Physician per policy C. Charge Nurse: -Round on identified patients with primary nurse to determine if additional monitoring or intervention is needed.
4	MEDIUM	If MEWS Score was a 2 or less from the previous recorded measure A. Licensed or non-licensed caregiver: -Notify RN of MEWS score B. Licensed caregiver: -Reassesses patients -Activate Rapid Response Team if clinical judgment deems necessary – Notify Physician per policy C. Charge Nurse: -Round on identified patients with primary nurse to determine if additional monitoring or intervention is needed.
5	MEDIUM	If MEWS Score was a 3 or less from the previous recorded measure A. Licensed or non-licensed caregiver: -Notify RN of MEWS score B. Licensed caregiver: -Reassesses patients -Activate Rapid Response Team if clinical judgment deems necessary – Notify Physician per policy C. Charge Nurse: -Round on identified patients with primary nurse to determine if additional monitoring or intervention is needed.
6 – 14	HIGH	A. Licensed or non-licensed caregiver: -Notify RN of MEWS score B. Licensed caregiver: -Reassesses patients -Activate Rapid Response Team if clinical judgment deems necessary – Notify Physician per policy C. Charge Nurse: -Round on identified patients with primary nurse to determine if additional monitoring or intervention is needed.